LOW SPEED AIR ATTACHMENT BASIC HANDPIECE MAINTENANCE



Remove bur from the attachment,

2

Wash the attachment under warm water and clean softly with a toothbrush.

After the proper E-Type nozzle for the attachment is connected, insert into the back end of the attachment and spray for 2 - 3 seconds or until lubrication is expelled from the head of the handpiece.



Manual



Care3 Plus™

Open the door and properly connect the attachment to the E-type adapter and make certain the bur hole is facing the door. Close the door securely.

Place the attachment upright to help drain the handpiece of excess lubricant. I5 minutes is sufficient. Or place attachment on the motor and with the head down run for I0 seconds.





If cleaning a speed reducer, select "EL" Mode for the extra long cleaning cycle. If cleaning a 1:1 attachment, select "L" Mode for the long cleaning cycle. Press the START button and wait until Care3 Plus completes cleaning, lubrication and purging cycle.



PERIODIC MAINTENANCE

Push-button chuck: Approximately once a week it is beneficial to spray the handpiece chuck for 3-5 seconds to insure optimum performance. Depress push button while spraying with PANA spray using pointed nozzle.



Bag and place handpiece in the autoclave. (Remember: Do not autoclave with bur in the chuck)

It is not necessary to spray the handpiece after the autoclave cycle is complete.

Handpiece questions?
Contact Brasseler USA directly!
1.800.841.4522 (ext. 4675)

By Your Side in Dentistry











