LS22K AIR MOTOR BASIC HANDPIECE MAINTENANCE



Wash the air motor under warm water and clean softly with a toothbrush.



Insert PANA spray tip into the drive air hole of the motor and spray for 2-3 seconds.



Purge the motor of excess lubrication either via an air station or by returning to the operatory and placing the motor back on the tubing. Purging excess lubricant for up to 10 seconds is sufficient.



Bag and place motor in the autoclave.

It is not necessary to spray the motor after the autoclave cycle is complete.

Handpiece questions?
Contact Brasseler USA directly!
1.800.841.4522 (ext. 4675)

By Your Side in Dentistry











